

A. V. Fistula Needles

WingEater™


Enable to prevent needle stick accidents from the end of dialysis until disposal by storing needle tubes in WingEater™

Instruction for storage



STEP1

Place gauze over puncture site. Hold gauze with your last three fingers while placing your index finger just behind the notch on the top of WingEater™.

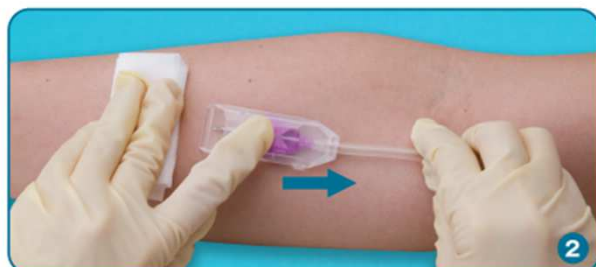

STEP2

Pull the tube (connecting tube) with the other hand and completely store the wing and needle tube in WingEater™.

SPECIFICATION


Product Number	Gauge	Wing Color	Needle length	Tube length	Package
621661221	16G	White	25 mm	30 cm	50 Pcs × 10 Boxes =500 Pcs
621761221	17G	Red violet	25 mm	30 cm	50 Pcs × 10 Boxes =500 Pcs
621861221	18G	Pink	25 mm	30 cm	50 Pcs × 10 Boxes =500 Pcs

A. V. Fistula Needles

SysLoc™ MINI


Enable to prevent needle stick accidents from the end of dialysis until disposal by storing and locking needle tube inside wing

Instruction for storage

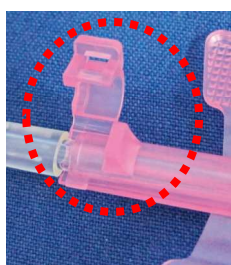

STEP 1

Hold the wing and the protrusion (of the side) while preventing the wing from rotating, then open the lock.

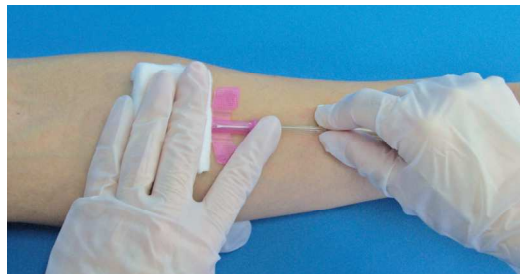

Stored needle

STEP2

Pull the connecting tube by the other hand and store up to the tip of the needle tubes' blades in the container.
*Insert it completely until you hear "click".

SPECIFICATION

Product Number	Gauge	Wing Color	Needle length	Tube Length	Package
621662302	16G	White	22 mm	30 cm	50 Pcs × 10 Boxes =500 Pcs
621762302	17G	Red violet	22 mm	30 cm	50 Pcs × 10 Boxes =500 Pcs
621862302	18G	Pink	22 mm	30 cm	50 Pcs × 10 Boxes =500 Pcs